SURFDIS

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www.sojadis.com

CONTENTS

SURFDIS's Overview	15
Usage	16
Maintenance	18
Warranty	19
Q&A	20
Let's drive together	21
Glossary	22

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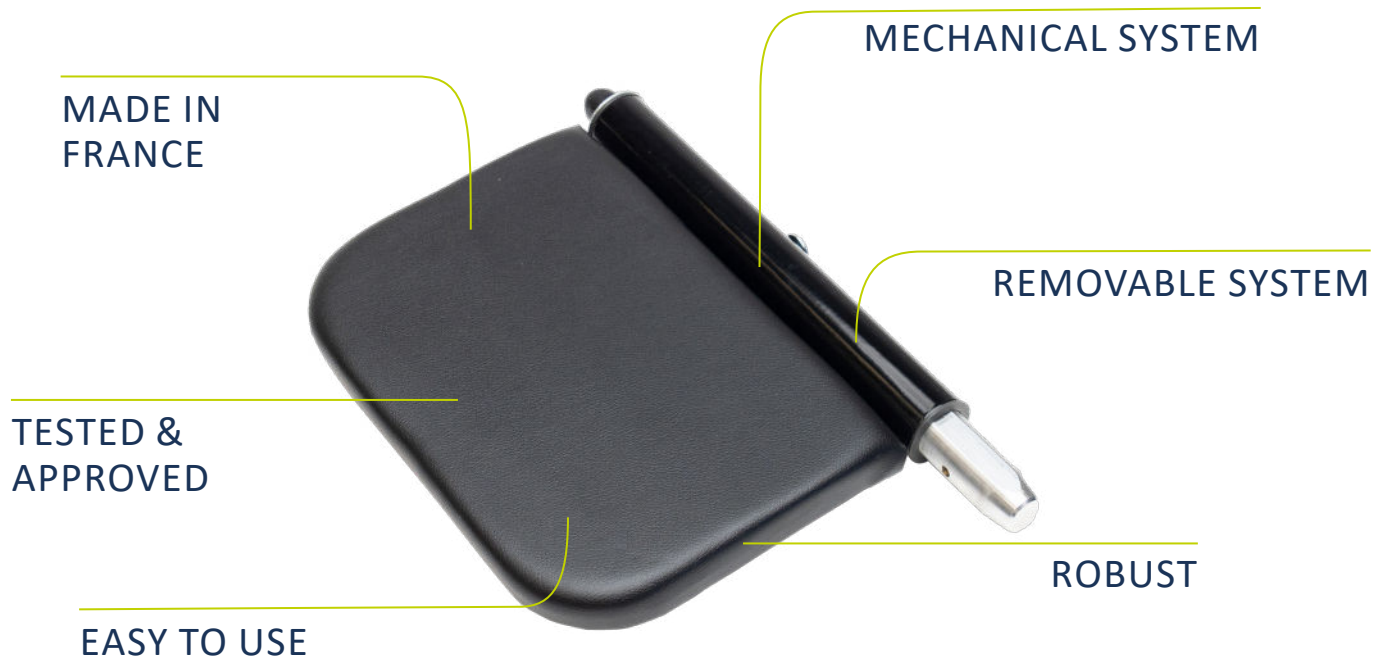
The Sojadis Team

SURFDIS'S OVERVIEW

COMPATIBLE WITH THE FOLLOWING PRODUCTS :

LEHMDIS

COMDIS

ACCELEVER

CERCLACCEL

ACCEL QUAD /
ACCEL INDEX

ACCEL BIKE /
ACCELEVO

USAGE

RECOMMENDATIONS

TECHNICAL SPECIFICATIONS

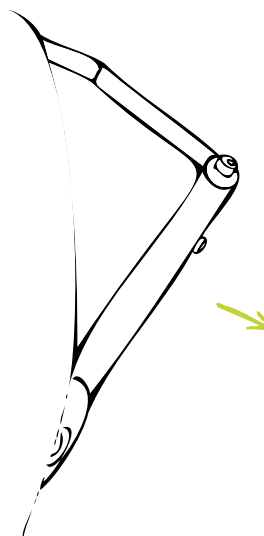
The maximum weight allowed on the board is 100 kg.

EASY TRANSFER


The SURFDIS allows you and your wheelchair to get as close to the vehicle as possible and transfer with ease.

INSTRUCTIONS

1 LOWER THE BOARD NEXT TO THE SEAT

2 SPIN THE BOARD

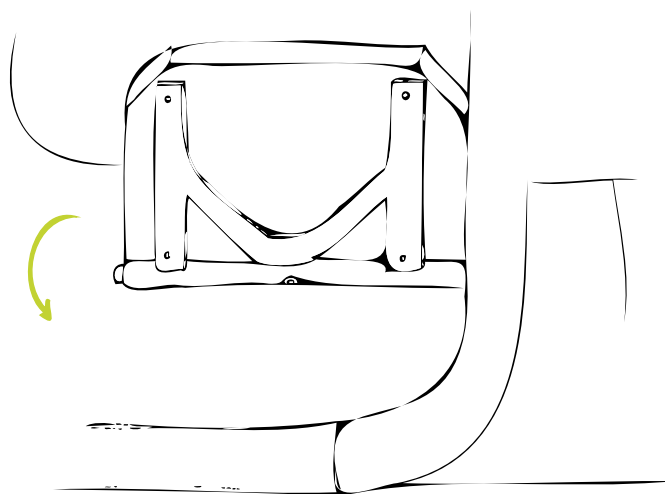3 COMPLETE YOUR TRANSFER

The transfer board must not pivot without the driver's intervention. If it lowers without you doing anything, please contact your installer.

STORAGE

After completing the transfer, it is advisable to store the transfer board alongside your seat.

STEPS TO FOLLOW:

- ① Spin the board.
- ② Fit the board between the seat and the vehicle pillar.
- ③ Lock the board using the safety catch. You should hear a «click», indicating that it has locked properly.

Ensure that you hear a click. This indicates that it is securely locked in place.

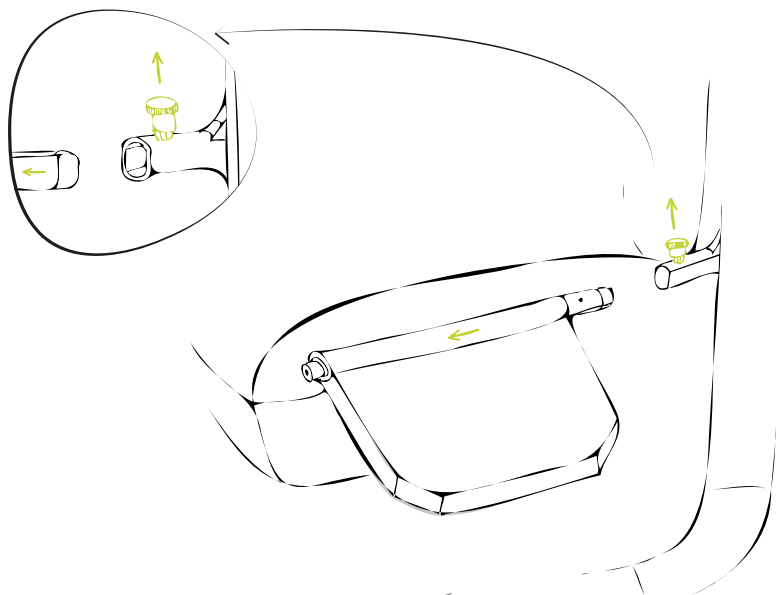
**WOULD YOU
PREFER A
VIDEO ?**

Click here or
scan the QR-Code.


ASSEMBLY AND DISASSEMBLY

MAINTENANCE

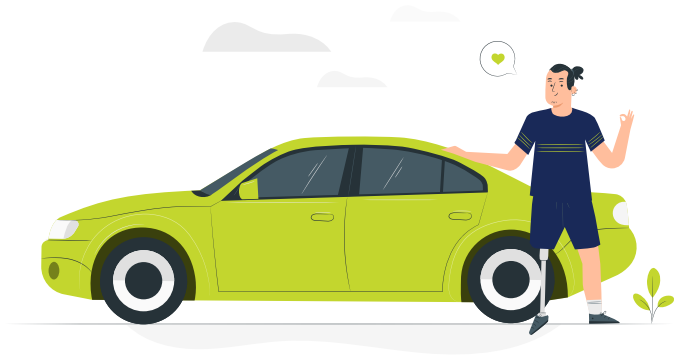OUR ADVICE

CLEANING

The SURFDIS requires no special maintenance.
Caution: Using products may cause damage to the functioning of the equipment.

DUSTING

To remove dust from the SURFDIS, use a soft microfibre cloth.

For all maintenance operations, contact our workshop or a Sojadis partner installer. ⁱ

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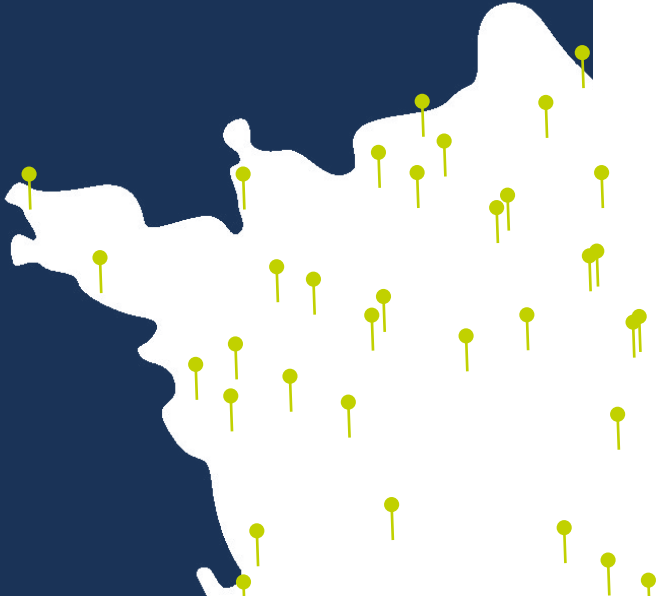
(via our contact form)

AUTODIAGNOSTIC

The transfer board lowers without any action on my part.

Contact your nearest Sojadis workshop or Sojadis partner installer.

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**LEAVE US
YOUR OPINION**

Your opinion counts.

Help us to improve
by sharing your
impressions.
Thank you !

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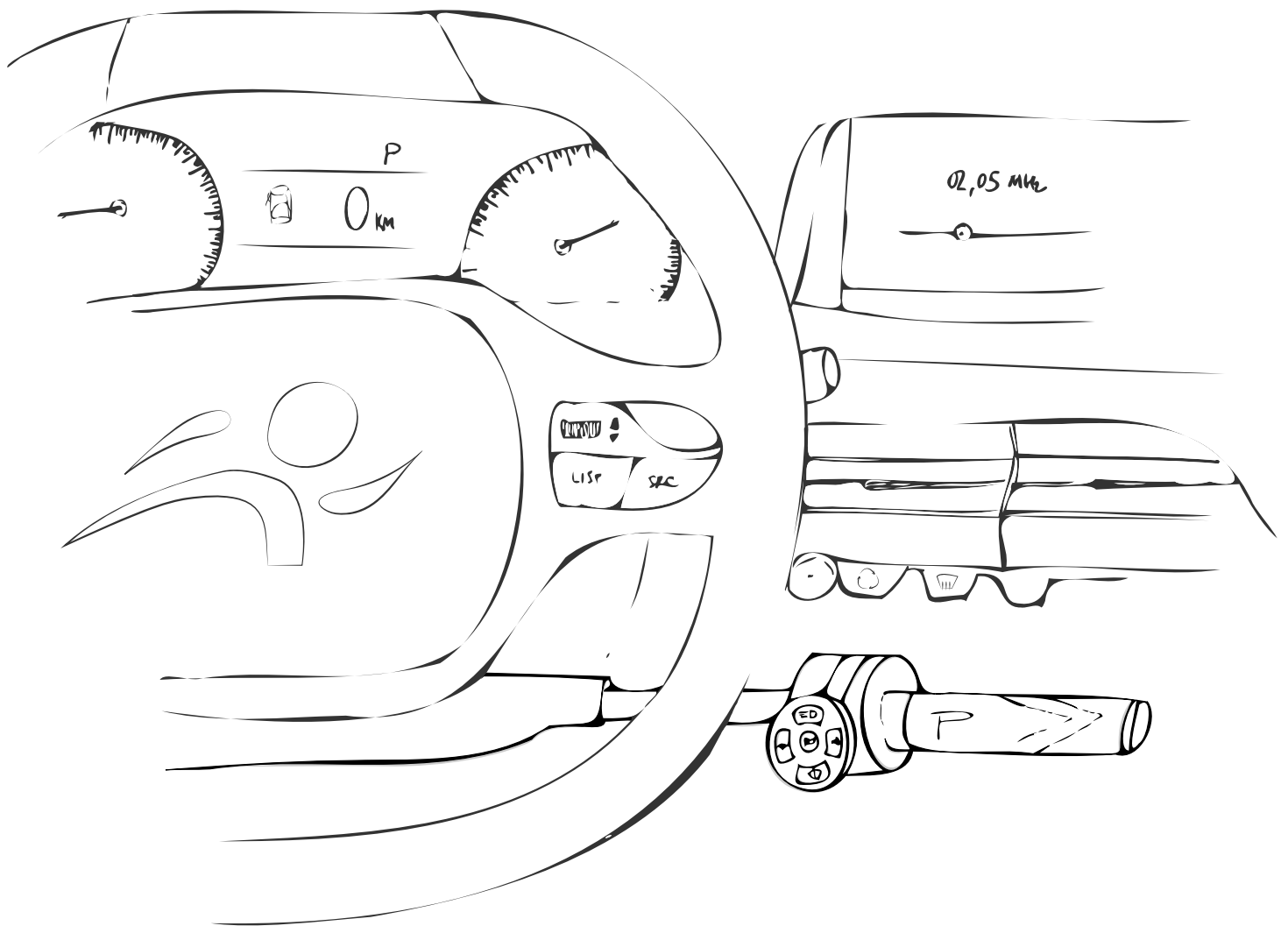
www.sojadis.com

ACCEL BIKE

MULTIFUNCTION ACCELERATOR
AND BRAKE HANDLE

sojadis
PLUG & DRIVE

www.sojadis.com

CONTENTS

ACCEL BIKE's Overview	15
Usage	16
Maintenance	18
Warranty	19
Q&A	20
Let's drive together	21
Glossary	22

Dear Customer,
You have just acquired a Sojadis product, and we thank you for your trust.
We wish you a safe journey with your vehicle and new equipment.

The Sojadis Team

ACCEL BIKE'S OVERVIEW

MINI KEYPAD (OPTIONAL) :

Horn

- On/off
Press
- Windshield washer
Hold press

Headlights

- Low beam
1st press
- High beam
2nd press
- Lights off
Hold press

Left turn signal

Right turn signal

- On/off
Press
- Brake lock
Hold press
(optional)

Windshield wipers

- Low speed
1st press
- High speed
2nd press
- To stop the wipers
Hold press

Backlit keyboard

USAGE

RECOMMENDATIONS

ACTIVATION

The equipment must be activated within 30 seconds. After that, the vehicle must be restarted and repeat the instructions.

When ACCEL BIKE is activated, the original accelerator pedal is deactivated.

START

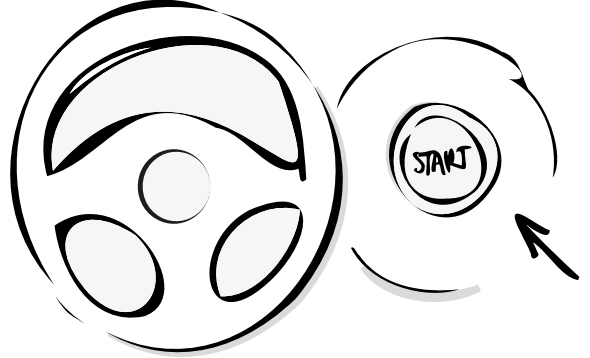
Do not press the ACCEL BIKE during vehicle start.

MINI KEYPAD


When the vehicle starts, the MINI KEYPAD is active, unlike the ACCEL BIKE, which remains inactive (no acceleration).

INSTRUCTIONS


1 START THE VEHICLE

2 PRESS THE ACTIVATION BUTTON

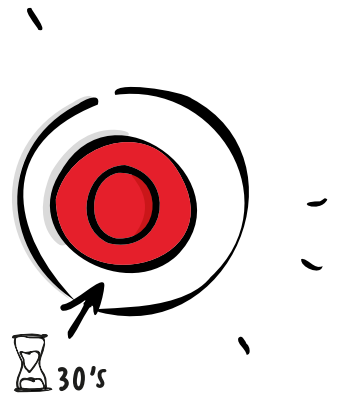3 USE YOUR EQUIPMENT

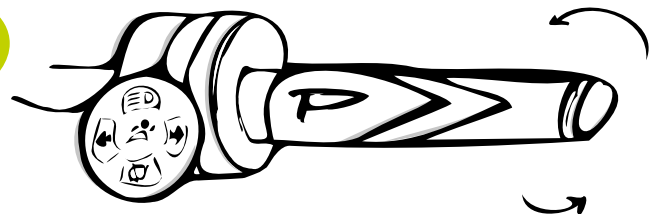PREFER A VIDEO ?

Click or scan the QR-Code.

SAFETY

Thanks to the electronics on board the ACCEL BIKE, braking automatically deactivates acceleration.

RETURN TO ORIGINAL CONFIGURATION

To return to the original configuration (adaptation equipment deactivated), switch off the engine and then restart the vehicle.

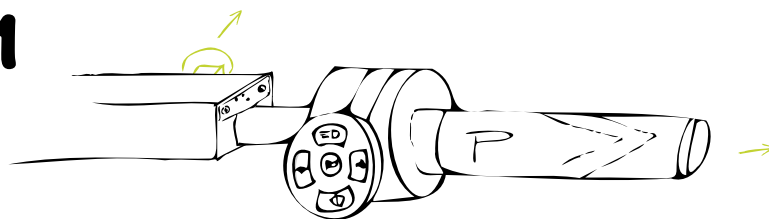
SPEED LIMITER

To deactivate the speed limiter (when overtaking, for example), press hard on your accelerator and pass the resistance point.


ASSEMBLY AND DISASSEMBLY (OPTIONAL)

1

2

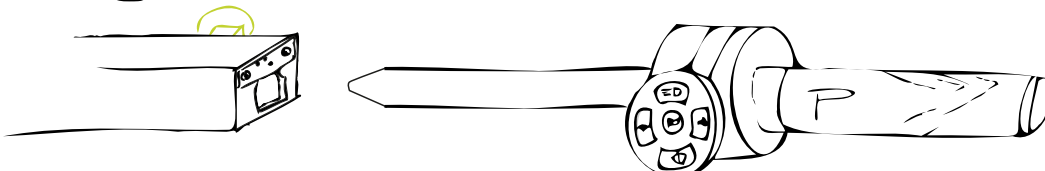MINI KEYPAD (OPTIONAL)

ACTIVATION & DEACTIVATION

When a function is activated using the MINI KEYPAD, it must be deactivated using the MINI KEYPAD.

When the vehicle's engine is switched off, the MINI KEYPAD controls in use are automatically switched off.

Depending on your configuration, the functions can be reactivated the next time the vehicle is started.

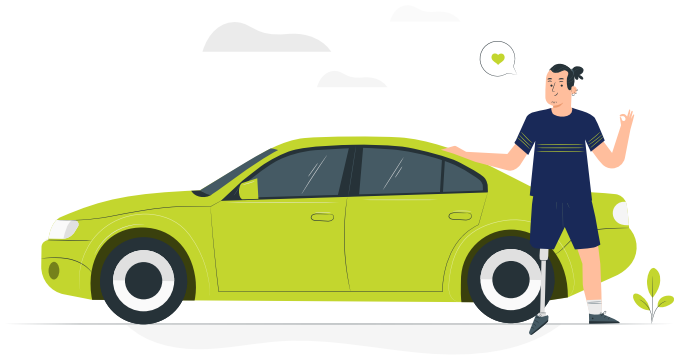
LOCKING THE BRAKE

Never pull on the brake lever to release the brake. Follow the locking/unlocking method on page 17 of the STOPDIS manual.

MAINTENANCE

OUR ADVICE

CLEANING

The ACCEL BIKE requires no special maintenance.
Caution: Using products may cause damage to the functioning of the equipment.

DUSTING

To remove dust from the ACCEL BIKE, use a soft microfibre cloth.

For all maintenance operations, contact our workshop or a Sojadis partner installer. 

WARRANTY

IN BRIEF

2
YEARS

This is the period during which the legal warranty applies.

The legal guarantee of conformity entitles you to have the product repaired or replaced. Contact your installation partner to find out how long this will take.

The warranty does not apply in the following cases: misuse of the product, accident, etc.

FIND OUT MORE

EQUIPMENT REMOVAL

Would you like to sell your equipped vehicle or remove your equipment?

Contact Sojadis or your local partner.

PRODUCT VALIDATION

Find out what we do to test and validate the reliability of our equipment throughout the design process.

Click here or scan the QR-Code

Q&A

AFTER-SALES SERVICE

Our after-sales service is at your disposal to diagnose, troubleshoot and find solutions to your problems. Please have your vehicle registration document ready in case of after-sales service.

+33 2 41 64 05 58

(Monday to Thursday from 8.30am to 6pm and Friday from 8.30am to 5pm)

sojadis@sojadis.com

www.sojadis.com

(via our contact form)

AUTODIAGNOSTIC

After starting my car, my equipment does not work

Check that the activation button (red button) is working (see operating instructions page).

The handle cover is slipping

Contact us or your nearest Sojadis installer.

The engine fault light appears on my dashboard

Contact us or your nearest Sojadis installer.

Handle jammed

Contact us or your nearest Sojadis installer.

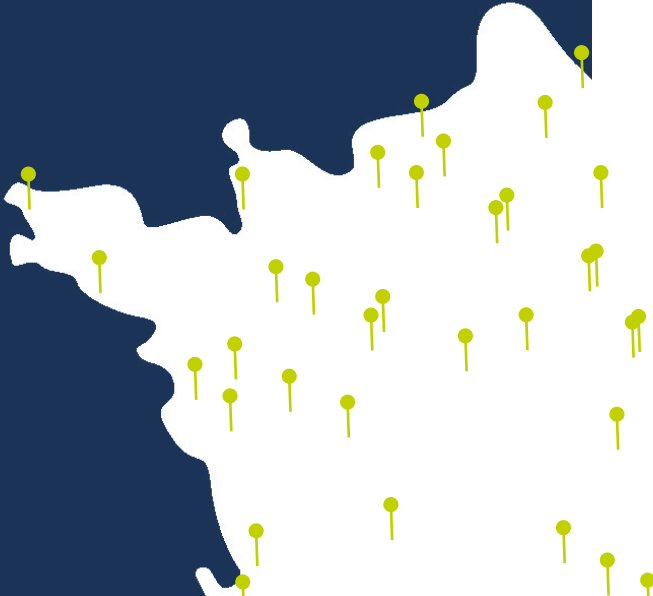
INSTALLER NETWORK

In the event of a breakdown, contact the Sojadis workshop directly or the Sojadis partner installer who fitted the vehicle.

PARTNER INSTALLER MAP

Click or scan
the QR-Code

LET'S DRIVE TOGETHER

SOJADIS

Taking your children to school, to a basketball match, meeting up with friends, going away for the weekend... We all aspire to be able to get around easily, to be able to cope with everyday life without depending on others, to be able to fulfil our potential as women or men - in short, to lead our own lives.

DRIVE YOUR LIFE* is the philosophy that drives the entire Sojadis team:

«Our company develops and manufactures technical solutions, embodying knowledge and expertise in cutting-edge electronics that are as rare as they are precious. But what adds value to this expertise is our desire to enable as many people as possible to drive in complete safety and with complete peace of mind, so that they can fulfil their life's plans.»

Adeline HUMEAU, President of Sojadis

*Drive Your Life is the Sojadis slogan.

**LEAVE US
YOUR OPINION**

Your opinion counts.

Help us to improve by sharing your impressions.

Thank you !

Click here or scan the QR-Code

Safetrip!

GLOSSARY

KICKDOWN

Kickdown is a function of automatic cars that reduces the gear down when the accelerator is fully depressed.

ACTIVATION BUTTON

The activation button turns red when the equipment is active. Note that this is not an on/off button. To deactivate the equipment, the vehicle's engine must be switched off.

SOJADIS PARTNER INSTALLER

Partner installers are car garages that have chosen to join the Sojadis network to offer their customers vehicle adaptations for people with disabilities and driving school conversions.

They are all trained and supported by Sojadis to guarantee the quality of the products and services offered. Thanks to this network, every customer can benefit from Sojadis' expertise and values, close to home.

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